

OBSERVER

**baby alarm and pager
Babyphon und Pager
alarma et garde bébé
alarma para bebé y localizador**



- ⓐ user guide
- ⓓ Bedienungsanleitung
- ⓕ instructions
- ⓔ instrucciones

)))))) **PhonicEar Logia™**

other resources online at www.phonicearlogia.com

OBSERVER function


OBSERVER is a wireless baby alarm and pager and one of the PUZZLE system's alerting transmitters. OBSERVER detects when a baby cries, then activates a red light on top of the unit and sends an alarm signal wirelessly to the PUZZLE system's receivers. There is also a button on top of the unit that lets you use OBSERVER as a pager. OBSERVER sends one alarm pattern when the baby cries and a different alarm pattern when used as a pager. It works with the following receivers in the PUZZLE system: CALL, FLASH, WAKE, POCKETVIB, LOOP, BEAM, and LINK.

OBSERVER features


- 1 built-in microphone
- 2 belt clip
- 3 personal code selector
Located inside battery compartment
- 4 alarm switch
To select alarm pattern.
Located inside battery compartment
- 5 battery compartment
- 6 sensitivity control
To adjust built-in microphone sensitivity level
- 7 indicator light
Light indicates when a signal is being transmitted by the OBSERVER unit
- 8 pager button
Allows you to use OBSERVER as a pager
- 9 power switch
Turns unit on and off

OBSERVER features


front

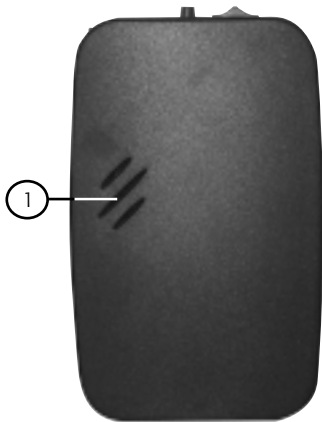

back

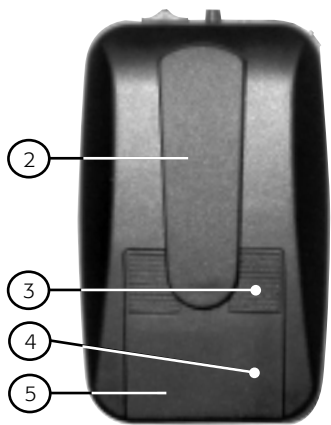

OBSERVER features

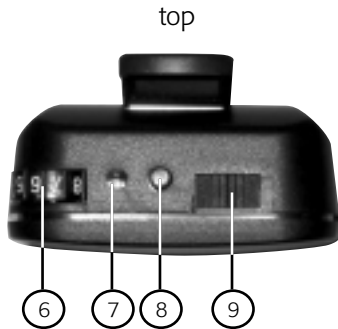

OBSERVER instructions

1 OBSERVER installation



OBSERVER should be placed as near as possible to the baby without being within reach.

NOTE: *OBSERVER should not be exposed to wet or damp conditions.*

OBSERVER instructions

2 operating OBSERVER



OBSERVER is switched on by sliding the switch on the top of the unit from left to right.

To turn it off, slide the switch from right to left.

OBSERVER instructions

3 selecting alarm patterns



If you have two OBSERVER units and wish to distinguish between the signals sent from each unit, the positions of the switches must be opposite.

If you have one or more DETECT alerting transmitters the selection of alarm patterns must be different for all DETECT and OBSERVER alerting transmitters.

	baby	pager
switch in left position	alarm 3	alarm 4
switch in right position	alarm 5	alarm 6

OBSERVER instructions

4 selecting personal code



The personal code can be selected using the tuner in the unit's battery chamber.

(See **6a** for instructions on removing battery compartment door).

NOTE: *The PUZZLE system's different alerting transmitters and receivers should be set to the same personal code.*

OBSERVER instructions

5a setting microphone sensitivity



1 Turn the microphone sensitivity control, on top of unit up to **8** (maximum sensitivity).

When receiving false alarms, turn the control one step toward **0**.

2 Repeat the above procedure until you only receive alarms when the baby is crying.

OBSERVER instructions

5b setting microphone sensitivity



NOTE: *Sensitivity should be adjusted as low as possible to minimize the possibility of false alarms.*

When red light on top of the unit lights, OBSERVER has detected a signal and is transmitting a signal.

OBSERVER instructions

6a changing batteries




Hold OBSERVER in both hands with back facing up and top facing toward you.

Press down on the marked areas with your thumbs and push the top away from yourself.

NOTE: *OBSERVER sends a signal to the PUZZLE system's receivers when the batteries need to be changed.*

OBSERVER instructions

6b changing batteries

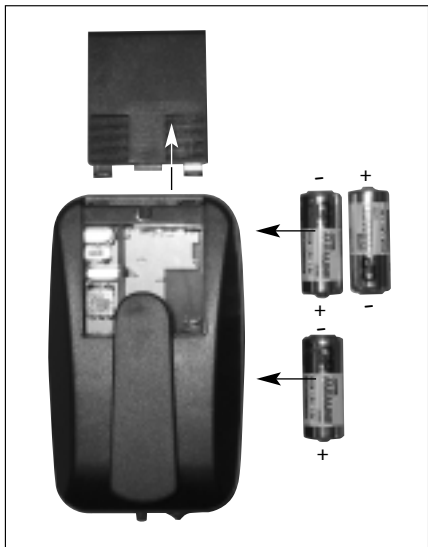

Insert new batteries, being careful to orient the batteries correctly according to the plus and minus symbols in battery chamber.

NOTE: Use three IEC LR1 penlight 1,5V batteries. The batteries should be removed if OBSERVER is not used for an extended period of time.

troubleshooting

Should your OBSERVER baby alarm and pager not function satisfactorily, please check the list below.

OBSERVER does not react to baby crying

- Check that the unit is turned on. See [2].
- Check that the microphone is adjusted correctly. See [5a] and [5b].

OBSERVER does not react to either the baby crying or pushing the pager button

- Check that the unit is turned on. See [2].
- Check that the batteries are correctly placed in the battery chamber. See [6b].
- Change the batteries in OBSERVER. See [6a] and [6b].

OBSERVER reacts to baby's cries and to pressing the pager button, but none or only some of the PUZZLE system's receivers give a signal

- Check that the personal code on OBSERVER and the PUZZLE system's receivers are the same. See [4].

troubleshooting (cont'd)

PUZZLE system's receivers activate an alarm without apparent reason

- Change the personal code using the tuner placed in the battery chamber. See 4.

If you still have trouble after trying the ideas above please contact your local authorized PhonicEar Logia dealer/installer.

tests and approvals

OBSERVER is approved and CE marked in accordance with the R&TTE directive.

warranty

A one-year limited warranty is issued to the original consumer of this product(s) by PhonicEar Logia.

time period of warranty

This warranty will go into effect upon the date of original purchase of this equipment, and will remain in effect for one calendar year from that date as long as the instrument remains the property of the original consumer.

what is covered by this warranty

Any electronic component which, because of workmanship, manufacturing, or design defects, fails to function properly under normal use during the life of this warranty will be replaced or repaired at no charge for parts or labor when returned to the factory service center. Transportation in is paid by the customer. If it is determined that repair is not feasible, the entire unit may be replaced with an equivalent unit upon mutual agreement of the manufacturer and customer.

what is not covered by this warranty

This one-year limited warranty does not apply to:

- 1** Malfunctions resulting from abuse, neglect, or accident.
- 2** Peripheral accessories as itemized within the product brochure, when such items are returned after 90 days from the original purchase.
- 3** Alkaline batteries (if applicable).
- 4** Instruments connected, installed, used, or adjusted in a manner contrary to instructions provided by the manufacturer.

warranty (cont'd)

- 5 Consequential damages and damages resulting from delay or loss of this instrument. The exclusive remedy under this warranty is strictly limited to repair or replacement as herein provided.
- 6 Products damaged in transit unless investigated by the shipper and returned to the warrantor with the investigation report.

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
PhonicEar Logia reserves the right to make changes in the design or construction of any of its instruments at any time without incurring any obligation to make any changes whatsoever on units previously purchased. This warranty is in lieu of all other expressed warranties. All expressed and implied warranties will terminate upon the expiration of this written warranty. No representative or person is authorized to represent or assume for us any liability in connection with the sale or use of our products other than as set forth above.

The above warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods. Your distributor may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult your distributor for further information.

what to do if you need service

If you require service under the terms of this warranty, carefully package the instrument to prevent damage and return it to your distributor (see information on back cover). Please provide your receipt, a detailed description of the problem, your full name, billing address, and telephone number.

Your local dealer:



europe: +45 3917 7101

germany (kostenlose Rufnummer): 0800/1844736

u.s.: 800.227.0735, then press 7 • **canada:** 800.263.8700

other regions: Call your local PhonicEar Logia dealer

website: www.phonicearlogia.com