

POCKETVIB

powerful wireless vibrator
kräftiger drahtloser Vibrationsmelder
vibreur sans fil puissant
potente vibrador inalámbrico



- ⓐ user guide
- ⓓ Bedienungsanleitung
- ⓕ instructions
- ⓔ instrucciones

)))))) **PhonicEar Logia™**

other resources online at www.phonicearlogia.com

POCKETVIB function


POCKETVIB is the PUZZLE system's vibrating receiver. Once an alarm from one of the PUZZLE system's alerting transmitters (DETECT and/or OBSERVER) is received, POCKETVIB vibrates while a diode on top of the unit also lights. POCKETVIB can receive up to seven different alarms apart from the alarm for battery change. Since each alarm has its own unique vibration pattern you can easily distinguish between the different alarms. The individual vibrations are selected on the alerting transmitter (DETECT and/or OBSERVER).

POCKETVIB features


- 1 indicator diodes
Diodes indicate which signal is being received by the POCKETVIB unit
- 2 belt clip
- 3 personal code selector
Located inside battery compartment
- 4 battery compartment
- 5 indicator light
Indicates when unit is turned on and a signal is being received
- 6 power switch
Turns unit on and off

POCKETVIB features

front

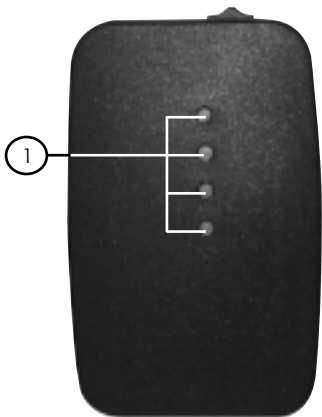


back

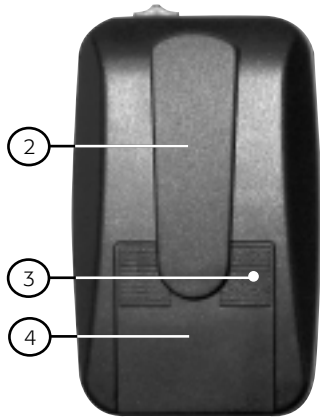

POCKETVIB features

top

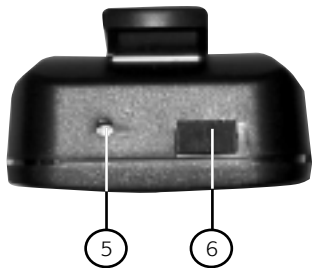

POCKETVIB instructions

1 turning on POCKETVIB



POCKETVIB is switched on by sliding the switch on top of the unit from left to right.

To turn it off, slide the switch from right to left.

POCKETVIB emits a signal to indicate that the unit is turned on.

POCKETVIB can be clipped to a belt, waistband, or the outside of a pocket.

POCKETVIB instructions

2 selecting personal code



The personal code can be selected using the tuner in the POCKETVIB battery chamber.

(See **4C** for instructions on removing battery compartment door).

NOTE: *The PUZZLE system's different alerting transmitters and receivers should all be set to the same personal code.*

POCKETVIB instructions

3a selecting alarm patterns


Each alarm has its own unique vibration pattern. The four diodes on the front of the unit also have different patterns denoting different alarms. The light and vibration patterns are set on the alerting transmitter (DETECT and/or OBSERVER).

| light diode | alarm | | | | | | | battery charge |
|-------------|-------|---|--------|---------|--------|---------|---|----------------|
| | 1 | 2 | baby 1 | pager 1 | baby 2 | pager 2 | 7 | |
| red | ● | ○ | ○ | ○ | ● | ○ | ○ | ● |
| yellow | ○ | ● | ○ | ○ | ○ | ● | ○ | ● |
| green | ○ | ○ | ● | ○ | ○ | ○ | ● | ○ |
| orange | ○ | ○ | ○ | ● | ● | ● | ● | ○ |

NOTE: *The diode light pattern stays on for 30 seconds after the alarm has been received.*

POCKETVIB instructions

3b recognizing alarm patterns

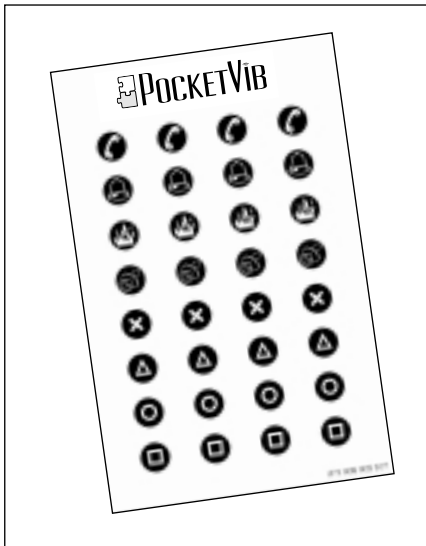

Enclosed is a sheet of stickers with illustrations of different alarms.

If you prefer, you can place a sticker next to the appropriate light diode.

For example, if the red light diode indicates the phone is ringing, place the handset sticker next to the red light diode.

NOTE: *Keep the extra stickers in case one falls off the unit and needs to be replaced.*

POCKETVIB instructions

4a low batteries




POCKETVIB emits a signal to let you know when the batteries need to be changed.

This signal is identical to the one it gives when it is switched on. See **1**.

POCKETVIB instructions

4b check POCKETVIB and OBSERVER batteries

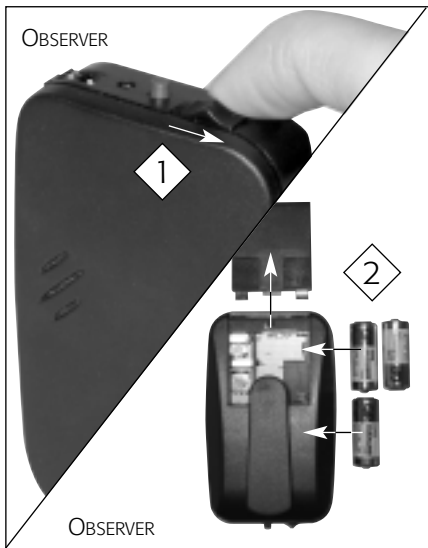


If you have an OBSERVER baby alarm and pager switched on when you get the battery warning, the low-battery alarm may be coming from OBSERVER.

- 1 Switch OBSERVER off and wait. If you hear a battery warning again then change the batteries in POCKETVIB. See [4C](#) and [4d](#).
- 2 If you do not hear another battery warning then change batteries in OBSERVER.

POCKETVIB instructions

4c changing batteries

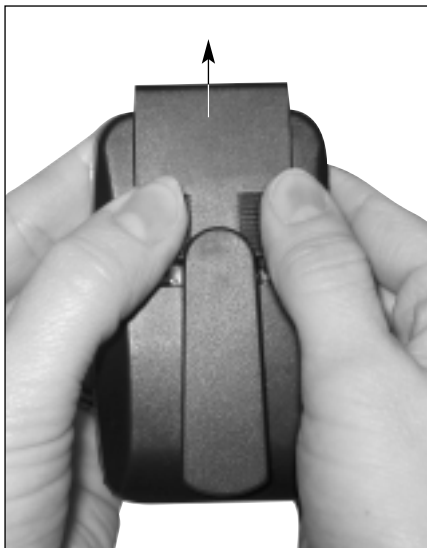


To change the batteries, hold POCKETVIB in both hands with the back facing up and the top facing toward you.

Press down on the marked areas with your thumbs and push the top away from yourself.

POCKETVIB instructions

4d changing batteries

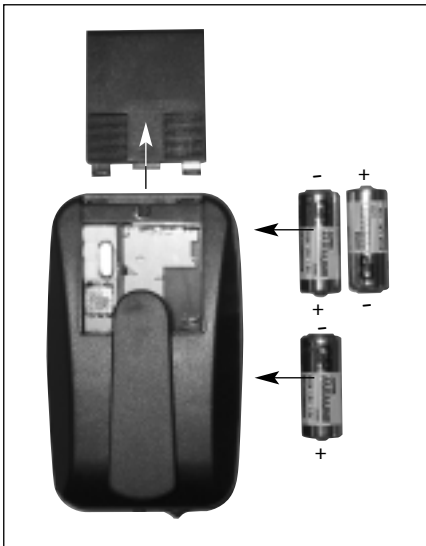

Insert the new batteries, being careful to orient the batteries correctly according to the plus and minus symbols in the battery chamber.

NOTE: Use three IEC LR1 Penlight 1.5V batteries. The batteries should be removed if POCKETVIB is not used for an extended period of time.

troubleshooting

Should your POCKETVIB not function satisfactorily, please check the list below.

POCKETVIB does not react when one of the PUZZLE system's alerting transmitters is activated

- Check that POCKETVIB is switched on. See [1].
- Check that the batteries are correctly placed in the battery chamber. See [4d].
- Check that the personal code on POCKETVIB and the PUZZLE system's alerting transmitters are the same. See [2].
- Change the batteries. See [4c] and [4d].

POCKETVIB emits a signal without apparent reason

- Change the personal code by using the tuner placed in POCKETVIB's battery chamber. See [2].

If you still have trouble after trying the ideas above please contact your local authorized PhonicEar Logia dealer/installer.

tests and approvals

POCKETVIB is approved and CE marked in accordance with the R&TTE directive.

warranty

A one-year limited warranty is issued to the original consumer of this product(s) by PhonicEar Logia.

time period of warranty

This warranty will go into effect upon the date of original purchase of this equipment, and will remain in effect for one calendar year from that date as long as the instrument remains the property of the original consumer.

what is covered by this warranty

Any electronic component which, because of workmanship, manufacturing, or design defects, fails to function properly under normal use during the life of this warranty will be replaced or repaired at no charge for parts or labor when returned to the factory service center. Transportation is paid by the customer. If it is determined that repair is not feasible, the entire unit may be replaced with an equivalent unit upon mutual agreement of the manufacturer and customer.

what is not covered by this warranty

This one-year limited warranty does not apply to:

- 1** Malfunctions resulting from abuse, neglect, or accident.
- 2** Peripheral accessories as itemized within the product brochure, when such items are returned after 90 days from the original purchase.
- 3** Alkaline batteries (if applicable).
- 4** Instruments connected, installed, used, or adjusted in a manner contrary to instructions provided by the manufacturer.

warranty (cont'd)

- 5 Consequential damages and damages resulting from delay or loss of this instrument. The exclusive remedy under this warranty is strictly limited to repair or replacement as herein provided.
- 6 Products damaged in transit unless investigated by the shipper and returned to the warrantor with the investigation report.

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
PhonicEar Logia reserves the right to make changes in the design or construction of any of its instruments at any time without incurring any obligation to make any changes whatsoever on units previously purchased. This warranty is in lieu of all other expressed warranties. All expressed and implied warranties will terminate upon the expiration of this written warranty. No representative or person is authorized to represent or assume for us any liability in connection with the sale or use of our products other than as set forth above.

The above warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods. Your distributor may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult your distributor for further information.

what to do if you need service

If you require service under the terms of this warranty, carefully package the instrument to prevent damage and return it to your distributor (see information on back cover). Please provide your receipt, a detailed description of the problem, your full name, billing address, and telephone number.

Your local dealer:



europe: +45 3917 7101

germany (kostenlose Rufnummer): 0800/1844736

u.s.: 800.227.0735, then press 7 • **canada:** 800.263.8700

other regions: Call your local PhonicEar Logia dealer

website: www.phonicearlogia.com